

MedVance® Foam Dressing With Border

Description

MedVance® Foam with border is a non-adherent polyurethane foam dressing that does not stick to wound bed. The porous structure increases absorbent capacity and speed while maintaining a moist wound healing environment. Breathable polyurethane film allows moisture vapor transmission, extending wear time, and the soft, comfortable foam may help redirect pressure. MedVance® Foam is also available without hydrocolloid adhesive border.

Indications for use

MedVance® Foam with border is intended for the management of moderate to heavily exuding acute or chronic wounds, including venous leg ulcers, diabetic foot ulcers, pressure injury wounds, donor sites and partial- and full-thickness burns.

Precautions

MedVance® Foam with Border is not appropriate for use on dry or lightly exuding wounds or those covered with dry necrosis. Do not use on patients with known sensitivity or allergy to polyurethane or hydrocolloid components. Product is provided sterile for single use. Do not use if package is open

or damaged. Do not re-sterilize.

Directions for use

1. Follow wound bed preparation protocol, including wound cleansing and debridement.
2. Select a suitable dressing size that overlaps the wound margin by approximately two centimeters.
3. Apply pad to the wound bed and smooth down adhesive border.
4. To remove, gently release adhesive border and remove dressing. Dispose as medical waste.
5. Follow wound bed preparation protocol, including cleansing with appropriate solution, before applying subsequent dressings.

Frequency of change

Dressing change frequency will vary as wound healing progresses. Daily dressing changes may be required initially until exudation decreases. Change when dressing is saturated, soiled or according to local protocol (up to seven days)

Warnings

Discontinue use and notify appropriate medical personnel if signs of infection are observed. Do not use with hypochlorite or hydrogen Peroxide solutions.

Storage

Store in a well-ventilated, cool, dry space.

Not made with natural rubber latex.



MedWay Inc.

1650 Horizon Parkway, Suite 450

Buford, GA 30518 U.S.A

Email: sales@medwayus.com

www.medwayus.com

Made In China

Revision: mv.ifu.1.1-v1