



MedVance® Silicone Foam Dressing With Border

Description

MedVance® Silicone Foam Dressing With Border is an absorbent, self-adherent island dressing consisting of polyurethane (PU) foam coated with soft and hypoallergenic silicone adhesive and breathable polyurethane film. The highly absorbent foam pad can absorb large amount of wound exudate, and the soft silicone adhesive acts as a gentle wound contact layer and border that sticks to the skin around the wound but does not stick to the wound bed, providing atraumatic and pain-free removal at dressing changes.

Indications for use

MedVance® Silicone Foam Dressing With Border is intended for the management of moderate to heavily exuding acute or chronic wounds, including venous leg ulcers, diabetic foot ulcers, pressure injury wounds, donor sites, and partial- and full-thickness burns.

Precautions

MedVance® Silicone Foam Dressing With Border is not appropriate for use on dry or lightly exuding wounds or those covered with dry necrosis. Do not use on patients with known sensitivity or allergy to the dressing components. Product is provided sterile for single use. Do not use if package is open or damaged. Do not resterilize.

Directions for use

1. Follow wound bed preparation protocol, including wound cleansing and debridement.
2. Select a suitable dressing size that overlaps the wound margin by approximately two centimeters.
3. Apply pad to the wound bed and smooth down adhesive border.
4. To remove, gently release adhesive border and remove dressing. Dispose of as medical waste.
5. Follow wound bed preparation protocol, including cleansing with appropriate solution, before applying subsequent dressings.

Frequency of change

Dressing change frequency will vary as wound healing progresses. Daily dressing changes may be required initially until exudation decreases. Change when dressing is saturated, soiled or according to local protocol (up to seven days).

Warnings

Discontinue use and notify appropriate medical personnel if signs of infection are observed. Do

not use with oxidizing solutions, such as hypochlorite solution or hydrogen peroxide solution (H₂O₂).

Storage

Store in a well-ventilated, cool, dry space.

Not made with natural rubber latex.



MedWay Inc.

1650 Horizon Parkway, Suite 450
Buford, GA 30518 USA
Email: sales@medwayus.com
www.medwayus.com
Made In China

Revision: mv.ifu.3.1-v1