



## MedVance® Collagen Dressings

### Product Description

MedVance® Collagen Dressings are porous collagen pads designed to be permeable and breathable, with fluid control similar to skin. They are flexible, pliable, elastic, and durable for ease of application. MedVance® Collagen Dressing are 100% non-hydrolyzed type 1 bovine native collagen. Proprietary technology processing protects and retains native triple helical protein structure allowing superior stability of the molecule and scaffolding through all four phases of wound healing.

### Indications for use

MedVance® Collagen Dressings are indicated for the management of burns, sores, blisters, scrapes, ulcers and other wounds.

### Precautions

- Do not use on third-degree (full thickness) burns.

- Do not use on patients with collagen or bovine sensitive.
- Single-patient use
- Sterile until opened
- Store at room temperature
- Do not use if package is damaged

### Directions for use

1. Cleanse wound as per physician order, preferable with normal saline or non-cytotoxic wound cleanser or wash.
2. Apply MedVance® Collagen Dressing to the wound bed surface.
3. Cover wound with a secondary dressing.
4. Change the dressing based on the drainage amount

### Storage

Store in a well-ventilated dry space between 50 – 85 °F.

Not made with natural rubber latex.



### MedWay Inc.

1650 Horizon Pkwy, Suite 450  
Buford, GA 30518 U.S.A

Email: [sales@medwayus.com](mailto:sales@medwayus.com)

[www.medwayus.com](http://www.medwayus.com)

Made in U.S.A

Revision: mv.ifu.7.1-v2