



MedVance® Collagen Powder

Product Description

MedVance® Collagen Powder is 100% non-hydrolyzed type 1 bovine native collagen. With a high surface area, allowing it to deliver the maximum amount of collagen to the wound surface. The proprietary production process technology protects and retains the native triple helical protein structure, allowing stability of the molecule and scaffolding through all four phases of wound healing.

Indications for use:

MedVance® Collagen Powder is indicated for the management of burns, sores, blisters, scrapes, ulcers and other wounds.

Precautions

- Do not use on third-degree (full thickness) burns.
- Do not use on patients with collagen or bovine sensitivity.

- Sterile until opened
- For single Patient use
- Do not use if package is damaged


Directions for use

1. Cleanse wound per physician order, preferably with normal saline or non-cytotoxic wound cleanser or wash.
2. Line wound bed with MedVance® collagen powder up to 1/4 inch depth.
3. Cover wound with a secondary dressing such as a transparent dressing, composite dressing or hydrocolloid.
4. Change the dressing as required for appropriate moisture management

Storage

Store in a well-ventilated dry space between 50 – 85 °F.

Not made with natural rubber latex.

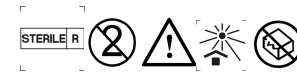

MedWay Inc.

1650 Horizon Pkwy, Suite 450
Buford, GA 30518 U.S.A

Email: sales@medwayus.com
www.medwayus.com
Made in U.S.A

Revision: mv.ifu.7.2-v2