

MedVance® Non-bordered Foam Dressing

Description

MedVance® Foam dressing is a nonadherent polyurethane foam dressing that does not stick to wound bed. The porous structure increases absorbent capacity and speed while maintaining a moist wound healing environment. Breathable polyurethane film allows moisture vapor transmission, extending wear time, and the soft, comfortable foam may help redirect pressure. MedVance® Foam is also available with hydrocolloid adhesive border.

Indications for use

MedVance® Foam dressing is intended for the management of moderate to heavily exuding acute or chronic wounds, including venous leg ulcers, diabetic foot ulcers, pressure injury wounds, donor sites and partial- and full-thickness burns.

Precautions

Do not use on patients with known sensitivity or allergy to dressing components. Product is provided sterile for single use. Do not use if package is open or damaged. Do not re-sterilize.

Directions for use

1. Follow wound bed preparation protocol including wound cleansing and debridement.
2. Select a suitable dressing size that overlaps the wound margin by approximately two centimeters.
3. Apply pad to the wound bed and apply a secondary dressing or bandage for fixation.
4. To change remove secondary dressing or bandage and remove dressing. Dispose as medical waste.
5. Follow wound bed preparation protocol, including cleansing with appropriate solution, before applying subsequent dressings.

Frequency of change

Dressing change frequency will vary as wound healing progresses. Daily dressing changes may be required initially until exudation decreases. Change when dressing is saturated, soiled or according to local protocol (up to seven days)

Warnings

Discontinue use and notify appropriate medical personnel if signs of infection are observed. Do not use with oxidizing solutions, such as hypochlorite solution or hydrogen peroxide solution (H₂O₂)

Storage

Store in a well-ventilated, cool, dry space.

Not made with natural rubber latex.



MedWay Inc.

1650 Horizon Parkway, Suite 450
Buford, GA 30518 U.S.A.

Email: MedVance@medwayus.com

www.medwayus.com

Made In China

Revision: mv.ifu.1.2-v1