



## MedVance® Super Absorbent Wound Dressing

### Product Description

MedVance® Super Absorbent Wound dressing is structured to absorb and retain significant amounts of wound exudate. The hydrophilic wound contact layer does not adhere to the wound site while transmitting fluid into the dressing. A distribution layer spreads exudate to maximize the use of the superabsorbent gelling properties within the pad enhancing fluid absorption and retention. The semi-occlusive backing layer prevents fluid and microbial strike-through while maintaining an appropriate moist wound healing environment.

### Indications for use

MedVance® Super Absorbent Wound Dressing is intended for the management of moderate to heavily exuding acute or chronic wounds including venous leg ulcers, diabetic foot ulcers, pressure injury wounds, donor sites and partial and full thickness burns.

### Precautions

MedVance® Super Absorbent Wound Dressing is not appropriate for use on dry or lightly exuding wounds or those covered with dry necrosis. Do not use on patients with known sensitivity or allergy to the dressing components. Product is provided sterile for single use. Do not use if package is open or damaged. Do not re-sterilize. Do not cut or use to pack cavity on undermined wounds.

### Directions for use

1. Follow wound bed preparation protocol including wound cleansing and debridement
2. Select a suitable dressing size that overlaps the wound margin by approximately 3-4 centimeters.
3. Apply secondary dressing, tape, or bandage to secure dressing in place.
4. When changing dressing, remove secondary dressing or bandage and remove dressing. Dispose as medical waste.
5. Follow wound bed preparation protocol including cleansing with appropriate solution before applying subsequent dressings.

### Frequency of change

Dressing change frequency will vary as wound healing progresses. Daily dressing changes may be required initially until exudation decreases. Change when dressing is saturated, soiled or according to local protocol (up to seven days)

### Warnings

Discontinue use and notify appropriate medical personnel if signs of infection are observed

### Storage

**Store in a well ventilated cool, dry space**

Not made with natural rubber latex.



### MedWay Inc.

1650 Horizon Pkwy, Suite 450  
Buford, GA 30518 U.S.A  
Email: [sales@medwayus.com](mailto:sales@medwayus.com)  
[www.medwayus.com](http://www.medwayus.com)  
Made In China

Revision: mv.ifu.6.2-v1