

MedVance® Silicone Tape

Product description

MedVance® Silicone Tape is designed to deliver enhanced conformability, extensibility, transparency, and reliable adhesion through the use of extra-thick medical-grade silicone adhesive under a thin polyurethane backing. MedVance™ Silicone Tape is waterproof, provides a low coefficient of friction, and tears easily for quick and practical application. Premium hypoallergenic silicone adhesive remains securely in place while allowing atraumatic, pain-free removal with no adhesive residue.

Indications for use

MedVance® Silicone Tape is intended for use as a secondary dressing to secure and seal primary dressings, tubing, cannula and other medical devices, and to protect skin from such devices as well as friction, shear and skin tears. Silicone has been employed to protect and reduce scars and to reduce MARSIS (medical adhesive related skin injury), pain and trauma on tape removal. Silicone adhesives allow for removal and repositioning of the tape.

Precautions

MedVance® Silicone Tape should not be used on patients with known sensitivity to its components.

Directions for use

1. Make sure the area is clean and dry prior to application.
2. Unroll and tear or cut a suitable length of tape.
3. Apply the adhesive surface to the skin and press to insure intimate contact with the skin.
4. To remove, raise corner and pull back and away from skin.

Frequency of change

MedVance® Silicone Tape may remain in place for several days to match primary dressing or device change frequency.

Warnings

Discontinue use if signs of infection or sensitivity develop.

Storage

Store in a well-ventilated, cool, dry space.

Not made with natural rubber latex.

MedWay Inc.

1650 Horizon Parkway, Suite 450
Buford, GA 30518 U.S.A.

Email: sales@medwayus.com

www.medwayus.com

Made in China

Revision: mv.ifu.3.3-v1

