



SILOVANCE

Curavance Nano Silver Gel Antibacterial Wound Dressing Gel

PRODUCT DESCRIPTION

SiloVance Antibacterial Wound Dressing Gel is a water-based gel for use in moist wound care management.

SiloVance Antibacterial Wound Dressing Gel contains silver that may help inhibit the growth of microorganisms within the dressing.

SiloVance Antibacterial Wound Dressing Gel is a water-based gel wound dressing for OTC use in the topical management of: first and second degree burns, lacerations, abrasions, minor cuts, skin irritations, and by or on the order of a licensed healthcare practitioner for: wounds such as stasis ulcers, pressure ulcers, diabetic ulcers, skin tears, surgical incision sites, device insertion site wounds, graft sites and donor sites.

SiloVance Antibacterial Wound Dressing Gel contains silver that in laboratory tests has been shown to inhibit the growth of microorganisms such as: *Staphylococcus aureus*, *Pseudomonas aeruginosa*, *Escherichia coli*, antibiotic-resistant bacteria, MRSA and VRE, as well as fungi such as *Candida albicans*.

INDICATIONS FOR USE

SiloVance Antibacterial Wound Dressing Gel is indicated for the OTC local management of: 1st and 2nd degree burns, lacerations, abrasions, minor cuts, and skin irritations.

And by or on the order of a licensed healthcare practitioner for the management of: wounds such as stasis ulcers, pressure ulcers, diabetic ulcers, device insertion site wounds, surgical incision sites, graft sites, donor sites, and skin tears.

Contents: 32 PPM Proprietary Silver (purified water, nano-silver at .01 micron), Propylene Glycol, Triethanolamine, Carbomer.

DIRECTIONS

Directions for use under the supervision of a licensed healthcare practitioner: SiloVance Antibacterial Wound Dressing Gel should be applied directly to the wound, and then covered with a conventional dressing. **SiloVance Antibacterial Wound Dressing Gel** should normally be changed every 24-72 hours, but the frequency will depend on the type and condition of the wound.

MANUFACTURED FOR:

Med Way Inc.
1650 Horizon Parkway, Suite 450
Buford, GA 30518
Email: sales@medwayus.com
Tel: (888) 5MEDWAY (633929)
www.medwayus.com
Made In U.S.A



SILOVANCE

Curavance Nano Silver Gel Antibacterial Wound Dressing Gel

Directions for OTC use (*not under the care of a licensed healthcare practitioner*): **1)** Wash your hands with soap and water before touching the wound. **2)** Carefully clean the wound using soap and water or an appropriate commercial brand of wound cleanser. **3)** Dispense **SiloVance Antibacterial Wound Dressing Gel** onto an appropriate clean applicator (such as a tongue depressor or gauze) in a sufficient amount to liberally cover the wound. Cover the gel with appropriate secondary dressing cover such as a gauze bolster, dressing wrap, film or non-woven adhesive secondary dressing.

Changing the Dressing: **1)** Carefully remove the secondary dressing from the wound. **2)** Residual **SiloVance Antibacterial Wound Dressing Gel** may be removed with the cover dressing. **3)** Follow instructions above to re-apply the gel.

Frequency of Change: **SiloVance Antibacterial Wound Dressing Gel** should normally be changed every 24-72 hours, but the frequency will depend on the type and condition of the wound.

PRECAUTIONS

Consult with a healthcare professional if you have any concerns or questions about the wound. Be alert for signs of wound infection. These can include: pus or cloudy fluid draining from the wound, increased redness around the wound, wound has become extremely tender, increased pain or swelling 48 hours after the wound occurred, onset of fever, and wound has not healed within 10 days from the date of the injury. **SiloVance Antibacterial Wound Dressing Gel** is designed for use on any part of the body except for oral and ophthalmic use. Consult a healthcare practitioner before using the gel in any other manner.

Intended for external use only. If your condition persists or gets worse, contact your healthcare provider.

Adverse Reactions: There are no known or reasonably suspected adverse reactions associated with the use of **SiloVance Antibacterial Wound Dressing Gel**.

Storage: Store at room temperature.

CAUTION: Federal law restricts this device to sale by or on the order of a licensed healthcare practitioner if it is used to treat the following: wounds such as stasis ulcers, pressure ulcers, diabetic ulcers, surgical incision sites, device insertion site wounds, skin tears, graft sites and donor sites.

MANUFACTURED FOR:

Med Way Inc.
1650 Horizon Parkway, Suite 450
Buford, GA 30518
Email: sales@medwayus.com
Tel: (888) 5MEDWAY (633929)
www.medwayus.com
Made In U.S.A